

AVAILABLE HERE

Strataderm®

scar therapy gel

## Strataderm - for the management of abnormal scars



### For old and new scars

- Used to soften and flatten raised scars
- Used to relieve the itching and discomfort associated with scars
- Used to reduce redness and discoloration



### Linear hypertrophic scars after surgery (nevus extraction)



Before treatment  
Age of scars: 14 months



Right scar treated for 2 months,  
left scar remained untreated



[us.strataderm.com](http://us.strataderm.com)

SD-US-050-1-0717

CE Class I Medical Device  
FDA Listed

Always read the label. Use only as directed.

Manufactured by: Stratpharma AG, Aeschenvorstadt 57, CH-4051 Basel, Switzerland  
Represented in the USA by: Stratpharma Inc., 1455 Frazee Road, Suite 500, San Diego, CA 92108, USA

 **Stratpharma**  
Switzerland

# Strataderm®

scar therapy gel

## How to use Strataderm

- Strataderm **should be applied once daily**, or twice daily to exposed areas or as advised by your physician.
- For best results Strataderm should be maintained in **continuous contact with the skin** (24 hours a day/7 days a week).
- Strataderm is recommended as a **minimum treatment of 60-90 days** (24 hours a day/7 days a week).
- Larger and older scars may take longer. **Continued use is recommended until no further improvement is seen.**
- Strataderm **does not soak into the skin**, rather it dries to form a very thin layer that is less than the thickness of a piece of paper or as if you stretch a piece of household clingfilm over your skin.




**1** Ensure that the scar is clean and dry.



**2** Apply a very thin layer of Strataderm to the scar and allow to dry. When applied correctly, Strataderm should be dry in 3-4 minutes.



**3** If it takes longer to dry you have probably applied too much. Gently remove the excess with a tissue and allow the drying process to continue.

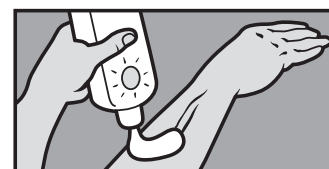

**4** Once dry, Strataderm can be covered by sunscreen, cosmetics, pressure garments or casts.



## How much Strataderm do I need?

Strataderm gel tube size	Scar size, which can be treated with one tube for over 90 days
0.17 oz (5 g)	linear scar, 1-2 inch (2-3 cm) length
0.35 oz (10 g)	linear scar, 2-3 inch (5-7 cm) length
0.7 oz (20 g)	linear scar, 4-5 inch (10-14 cm) length
1.75 oz (50 g)	widespread scar, (2.5×5 inch) (6×12 cm) area



[us.strataderm.com](http://us.strataderm.com)

SD-US-050-1-0717

CE Class I Medical Device  
FDA Listed

Always read the label. Use only as directed.

Manufactured by: Stratapharma AG, Aeschenvorstadt 57, CH-4051 Basel, Switzerland  
Represented in the USA by: Stratapharma Inc., 1455 Frazee Road, Suite 500, San Diego, CA 92108, USA

 **Stratapharma**  
Switzerland